



focus on health

Aluminium Spray

Liquid bandage

200 ml aerosol



Distributed by:
Anupco Ltd T/A Kela Animal Health UK
Tel: 01473 822121 email: sales@kela.health
www.kela.health

Aluminium Spray

Unprotected injuries can become rapidly infected. With the Aluminium-Spray the injury site is covered by an elastic film that allows air to pass through. The spray-on dressing protects against soiling and other harmful environmental influences.

As a result of its ease of application, wounds that can only be accessed with difficulty can also be protected.

Ingredients

Ethyl acetate, polyisobutyl methacrylate, aluminium powder, white mineral oil.

Propellant: butane

Application

Shake well before use.

Spray the wound area lightly from a distance of 20 - 25 cm - free from draughts - for 1 to 2 seconds. If necessary, repeat several times daily.

Package size

200 ml aerosol

Notes

For external application only.

Avoid contact with the eyes.

Extremely flammable aerosol. Pressurised container: May burst if heated. In contact with water releases flammable gases. Causes serious eye irritation. May cause drowsiness or dizziness. Harmful to aquatic life.

If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.

No smoking. Do not spray on an open flame or other ignition source. Handle and store contents under inert gas. Protect from moisture. Do not pierce or burn, even after use.

Brush off loose particles from skin. Immerse in cool water/wrap in wet bandages.

In case of fire: Use sand, carbon dioxide or powder extinguisher to extinguish.

Store in a well-ventilated place. Keep container tightly closed. Protect from sunlight.

Do not expose to temperatures exceeding 50 °C.

Manufacturer

Veyx-Pharma GmbH
Soehreweg 6
34639 Schwarzenborn
Germany
Phone 0049 5686 99860
E-Mail zentrale@veyx.de
www.veyx.com

Distributor for UK

Anupco Ltd T/A Kela Animal Health UK
39 Lodge House, Lodge Park, Lodge Lane, Langham
Colchester, Essex, CO4 5NE
sales@kela.health
www.kela.health

The information given in this catalogue sheet corresponds to the state of knowledge upon completion. Please, kindly thoroughly read the label prior to use.

